

# UV Treated N95: Donning & Doffing

## PREPARE BAG & MASK

- 1 Reverse crease sides of bag so it remains open.




- 2 Label the bag:
- Full Name
  - Unit/Dept. return location



- 3 Label N95 Respirator with Sharpie:

- First Initial & Last Name
- Date of first use
- Department location

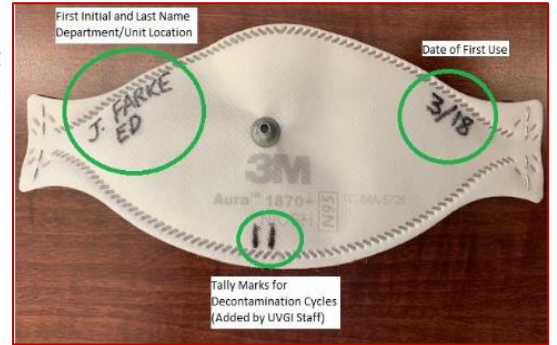

## Redonning UV Disinfected N95

- 1 Perform hand hygiene.



- 2 Don gown.



- 3 Don the N95 respirator. Perform a negative/positive seal



- 4 Don face shield or goggles



- 5 Perform hand hygiene.



- 6 Don gloves.



# UV Treated N95: Donning & Doffing

## DOFFING

1 Perform hand hygiene over gloves.



2 Remove gown & gloves in patient room.



3 Exit patient room.




4 Perform hand hygiene and don clean gloves.



5 Remove goggles or face shield and disinfect.



6 Store eyewear.

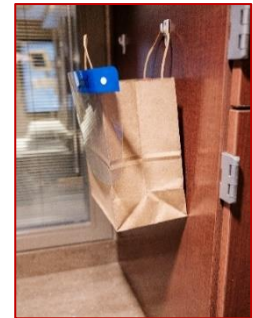

7 Perform hand hygiene on gloves.



8 Remove N95 and place into brown "dirty" bag.



9 Remove gloves and perform hand hygiene.

